

2026/09/16 15:01 1/1


Note that when Repeat is selected, the playback will be continuous, i.e., after the last frame, the playback continues with the first frame. The Playback Speed controls the speed of the playback relative to real time.

Motion Playback

References

After loading a motion into PlanDSX from a CMZ file, the motion capture model can be controlled using the playback widget located under the 3D view.

[Back to PlanDSX Overview](#)

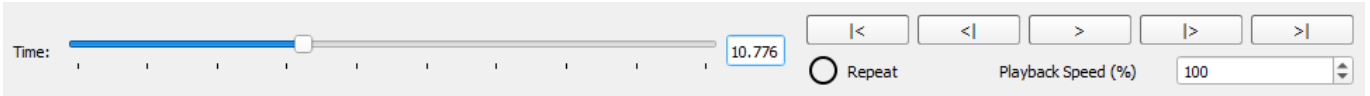

You can drag the slider to play back some of the frames, or use the buttons on the right to control the animation.

<https://has-motion.com/wiki/> - **Wiki Documentation Site**

Permanent link:

https://has-motion.com/wiki/doku.php?id=other:dsx:plandsx:motion_playback&rev=1736544952

Last update: **2025/01/10 21:35**

